Smith & Nephew Record	
Item:	Opsite Spray leaflet
Code:	66000679
This Program:	16900
Prev Asset:	14979
Date:	22.07.2018
This Revision:	00

ADAMS DESIGN Record	
Job No:	R003
Dims:	148.5 x 140mm - Prints both sides - Finished size 148.5 x 24mm
Colour	■ Black

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16900

Description

OPSITE Water Resistant Spray-On Dressing produces a clear film dressing, which allows the skin to breathe. It is ideal for unbroken blisters, minor burns and scalds. It is a simple and easy to apply spray, protecting the wound with a film as thin and comfortable as a second skin.

Indications

OPSITE Water Resistant Spray-On Dressing is designed for use on unbroken blisters, minor burns and scalds.

Contra-Indications

Do not use on children under the age of 18 months.

Instructions for use

1. Wash hands before and after treating the wound.
2. Gently clean and dry the wound and surrounding skin.
3. Shake well, invert can and spray from a distance between 15-20cm (6-8 inches) away from skin. Spray intended area with an even layer. For best results build up the dressing with a number of thin layers.

Note: A stinging sensation may be felt when first applied to broken skin.

Dressing Care

OPSITE Water Resistant Spray-On Dressing can be left in place for several days.

The thin layers can be peeled off when dry or left to fall away over time. An adhesive remover may be used to help removal.

Precautions for use

- Keep out of the reach of children.
- For external use only.
- Avoid inhaling. Use in a well ventilated area.
- Avoid contact with EYES. In case of contact with eyes, rinse immediately with lots of water and seek medical advice.
- Skin reddening may occur when used with medicated preparations.
- Avoid spraying onto fabrics.
- Keep away from naked flames or direct heat as contents are inflammable. No smoking.
- Do not spray on a naked flame or any incandescent material.
- This container is pressurised. Do not incinerate or expose to direct heat even when empty.
- This product contains acetone, which has the potential to react with non-latex/synthetic gloves.

Product Availability

OPSITE Water Resistant Spray-On Dressing is available in the following size: 40ml.

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Smith & Nephew Medical Ltd, 101 Hessle Road, Hull HU3 2BN England. *Trade Mark of / Marchio di / Marca comercial de Smith & Nephew. ©Smith & Nephew Date of issue / Data di revisione / Fecha de emisión 06/2018. Made in Germany / Prodotto in Germania / Fabricado en Alemania